

INSTRUCTIONS

- Turn off your device and disconnect power before disinfecting.
- Never use spray or chemicals directly on the unit.
- Disinfect all parts for your device, including cradle and charger.
- Allow your unit to air dry after disinfecting.

INFO

- Use only the recommended disinfection products for your device.
- In your device's manual you will find recommended disinfectants.
- HealthCare models can withstand certain hydrogen peroxide solutions.

CAUTIONS

- Certain chemicals can damage the plastic on the device.
- Be extra careful with bleach products.
- Some hand disinfection products contain ingredients that are harmful to the device.



Delfi Technologies
E-mail: info@delfi.com
Web: www.delfi.com



DISINFECT YOUR DEVICES

Prevent infection by
disinfecting your devices

